

MacroKnife

Instructions For Use (IFU)


Please select your language of choice from the list below.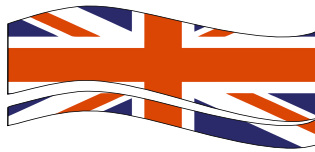

EN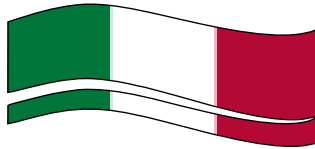

IT

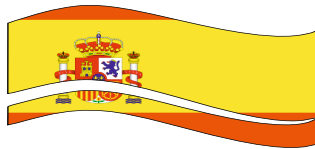ES

Black Diamond Knives™ / MacroKnife™

Disposable Pathology Knives

CAA-0401-01A - 4 Inch (100mm) Cartilage Knife


CAA-0601-01A - 6 Inch (150mm) Trimming Knife

CAA-0801-01A - 8 Inch (200mm) Macro Knife

CAA-1001-01A - 10 Inch (250mm) Brain Knife

CAA-1201-01A - 12 Inch (300mm) Lung Knife

CAA-9901-01A - Assorted Knives (2 x Each Size)

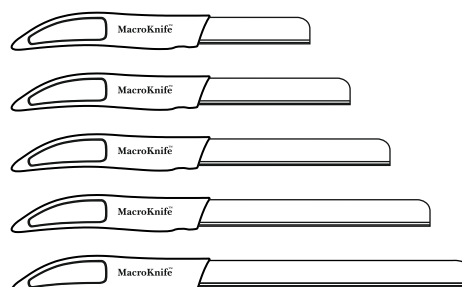INTENDED USE:

Black Diamond (AKA MacroKnife) pathology knives, are disposable ergonomic knives designed for cut-up/grossing of specimens derived from the human body. The freshly grossed specimen, is used to facilitate the diagnosis of the pathological state, and is intended for *in-vitro* examination of cellular deformity.

PRINCIPLE OF THE METHOD:

Black Diamond Knives, are designed to provide a safe and cost effective alternative to re-sharpening of knives used for cut-up and post mortems. The Black Diamond Knives are available in five sizes 100mm (4 inch), 150mm (6 Inch), 200mm (8 inch), 250 (10 Inch) and 300mm (12 Inch) which enables a pathologist to select the optimum size of knife to meet their cutting requirements.

Complete disposal of the knife and handle eliminates any health and safety concerns associated with knives where a replacement blade is fitted within a re-usable handle. It also eliminates the risk of cross contamination when working with 'high risk' specimens.

INSTRUCTIONS FOR USE:

Always use in a slicing motion, on soft tissue only.
Do not use with a hacking motion.
Use in conjunction with a chain mesh glove(s).
Do not re-sharpen.
Do not use Phenolic Disinfectant.
Dispose of all used knives in a sharps container.

SAFETY INSTRUCTIONS:

Please refer to either these Instructions For Use (IFU) or product packaging, for information regarding precautions when handling and disposing of this product.

The Black Diamond Knives are for medical use only, and are intended for *in-vitro* examination of human tissue, therefore they should only be handled by trained biomedical, scientific or medical staff where appropriate.

Black Diamond Knives are extremely sharp, use care when handling. Use of cut resistant gloves is recommended when using 'Black Diamond Knives'.

STORAGE INSTRUCTIONS:

This product should be stored at room temperature away from damp conditions.

TYPE OF HISTOLOGICAL MATERIAL TO BE USED:

Any fresh or fixed human tissue specimen for pathological examination.

If a new chemical solution (fixative) is used for fixation of a specimen, tests should be undertaken, to ensure that the solution has no detrimental effect on the integrity of the Black Diamond Knife. The Black Diamond Knives are not supplied as a sterile product.

PERFORMANCE CHARACTERISTICS AND METHOD LIMITATIONS:

- Five sizes of Black Diamond Disposable Pathology Knives are available, which can be selected according to the size of the tissue specimen which requires grossing.

100mm (4 inch) Cartilage Knife
150mm (6 inch) Trimming Knife
200mm (8 inch) Macro Knife
250mm (10 inch) Brain Knife
300mm (12 inch) Lung Knife

- The Black Diamond Disposable Pathology Knives are manufactured using high quality Sheffield Stainless Steel double honed blades with an integral injection moulded handle with finger guard. Flexibility of blades provides precise control and fine slicing ability.

If a Black Diamond Knife is used during cut-up of a 'high risk' specimen, the knife must then be considered contaminated, and should be disposed of according to your laboratory protocols.

RECOMMENDATIONS:

The correct size of Black Diamond Disposable Pathology Knife, should be chosen according to the size of the tissue specimen which requires grossing.

Check both the cut-up board and Black Diamond knife for dirt, cracks and debris, prior to use.

Black Diamond Knives are designed to be used in a slicing motion on soft tissue.

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CellPath Ltd