CerviBrush+ Profile LBC - Cervical Sampler & CerviBrush+ LBC (Endocervical Sampler)

NCA-0422-02A, NCA-0780-01A, NCA-0780-02A

EC	REP
----	-----

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

CellPath 
INNOVATION IN CELLULAR PATHOLOGY

 **CellPath Ltd**
80 Mochdre Enterprise Park,
Newtown, Powys, SY16 4LE, UK
T: +44 (0)1686 611333
E: sales@cellpath.com
cellpath.com

CerviBrush+ Profile LBC - Cervical Sampler & CerviBrush+ LBC (Endocervical Sampler)

NCA-0422-02A, NCA-0780-01A, NCA-0780-02A

A device with bristle-like projections designed to obtain a cervical biopsy of a suspicious area or visible exocervical lesions detected during a vaginal examination for the purpose of obtaining a tissue diagnosis from a woman with intraepithelial disease. It is inserted into the vagina and the head of the brush is firmly rotated over the area of interest until it is covered with a bloody-mucoid sample. Some types may also have small bristles on a central protrusion that simultaneously enters the endocervical canal. The sample is secured in a cytology fixation solution or smeared onto a microscope slide for a Papanicolaou test (PAP). This is a single-use device.

For professional use.

Store in a dry place. Do not store in direct sunlight. Do not use if packaging is damaged.

Please see expiry on product label.

Instructions for Use

1. Remove the plastic straw that protects the LBC head and discard (CerviBrush+ Profile LBC) [Pic 1].

2. Insert the LBC gently into the endocervix and rotate one half to one whole turn ensuring enough sample is taken. [Pic 2].

FOR THINPREP USE:

Rinse the CerviBrush+ in the vial by pushing the bristles against the vial wall to release the cells. To release additional cells, swirl the brush around. Then discard the brush and send vial to laboratory.

Isolate cells from preservative and transfer onto slide. Dispose of vial.

FOR SUREPATH USE:

Remove the CerviBrush+, break off the brush head and put into cyto sample vial. Send sample to laboratory. Centrifuge vial to separate cells from preservative and transfer cells onto slide. Dispose of vial and CerviBrush+ head.

CONVENTIONAL USE:

Remove the CerviBrush+ and prepare the smear by rolling and twisting the brush on a clean, labelled glass slide [Pic 3]. If desired, more than one slide can be prepared before disposing of the brush. Smear preparation should be fixed immediately with CytoFixx or similar fixative [Pic 4].

STERILISATION:

If samples are needed for microbiological examination (e.g., Chlamydia), the CerviBrush+ can be sterilised by Ethylene Oxide if desired, however it is only intended to be used once then discarded after preparing the smear.

CAUTION:






DO NOT use for smear sampling the endometrium.

DO NOT use if pregnancy is suspected or confirmed. Use the Aylesbury or a conventional spatula.

Dispose of in compliance with all local and national regulations.

DO NOT REUSE THIS PRODUCT.

Any complaints or adverse incidents need to be reported to CellPath immediately.

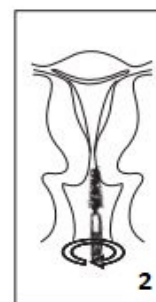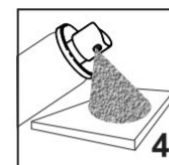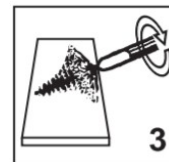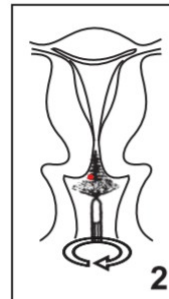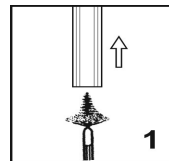CerviBrush+ Profile LBC - Cervical Sampler & CerviBrush+ LBC (Endocervical Sampler)

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