

INSTRUCTION FOR USE

ROMED PEDIATRIC URINE BAGS


STERILIZED BY EO-GAS (REF: PEDUB-100)

Paediatric urine bags

Sterile and made of polyethylene material, graduated at 10 ml intervals, strong and hypoallergenic adhesive, transparent, simple to use, for collecting urine from children (male & female) who cannot voluntarily control urination. This bag fits for the children body contours precisely so that the possibility of faeces contamination is prevented. Individually packed and sterilized by ethylene oxide gas.

Instruction for use:

1. Expose the adhesive surface by removing paper backing.
2. For male use, the penis to be positioned through the opening in the bag.
3. For female use, the opening in the bag to be placed in such a way that upper half of the genitalia are covered.
4. When the urine is collected, remove the bag and fold the two adhesive surfaces together, the bag becomes a sealed waterproof container.

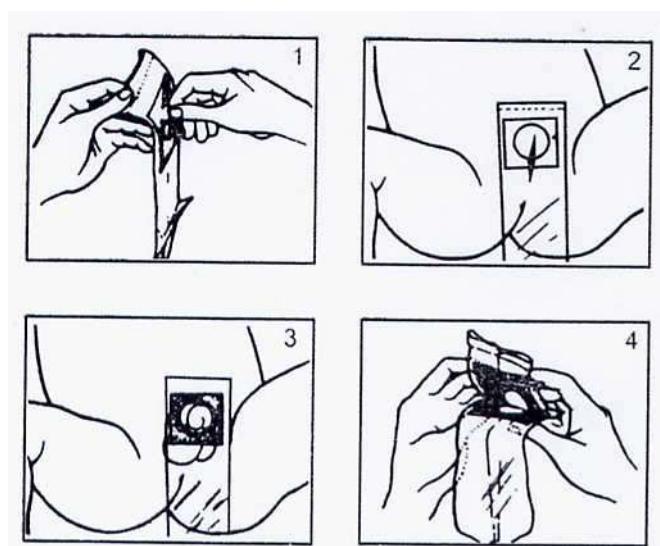

Caution:

After sterilization with EtO gas, use within valid time according to the valid date on the primary bag.

- Set is ready for use.
- Open and use immediately.
- Discard after use.
- Do not use if package is previously opened.
- Do not store under direct sunlight or extreme temperature.

VAN OOSTVEEN MEDICAL B.V. Romed®-Holland
Herenweg 269, 3648 CH Wilnis – HOLLAND
E-mail : info@romed.nl / Website www.romed.nl

Revision date October 2016

CE
0044